

pts connect™ dock



User Guide

Intended Use

The PTS Connect™ dock system is intended for use by professional customers. The PTS Connect dock system allows A1CNow® system and PTS Detect™ analyzers to transfer the most recent test result to a connected computer.

Hardware Requirements

Included:

- PTS Connect dock
- USB to RS232 Serial Adapter (Adapter)
- USB to USB cable








Required but not included:

- Computer with available USB port

Operation

Perform the following steps to transmit the very last test result from analyzer to a computer equipped with an available USB port:

1. Connect the male 9-Pin connector of the supplied adapter to the PTS Connect dock female 9-Pin connector.
2. Connect the USB connector of the supplied cable to the dock and the other end to an available port on the computer.
3. Run an application which captures results from the PTS Connect dock.
4. Run a test on an A1CNow analyzer or PTS Detect analyzer.
5. Once testing is complete, and the result is displayed on the screen, remove the test cartridge from the analyzer.
6. Place the analyzer on the cradle of the PTS Connect dock and press it down when ready to download data.
7. The analyzer will display the word "DOCK" and the result will be transferred to the application.
8. Remove the analyzer from the PTS Connect dock.
9. The PTS Connect dock is ready for the next test result to be transferred.

EXPLANATION OF SYMBOLS	
	CATALOG NUMBER
	CONSULT INSTRUCTIONS FOR USE
	AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	LOT NUMBER
	MANUFACTURER
	THIS PRODUCT FULFILLS THE REQUIREMENTS OF EUROPEAN DIRECTIVE 98/79 EC FOR IN VITRO DIAGNOSTIC MEDICAL DEVICES
	SEPARATE COLLECTION: BATTERIES MUST BE DISPOSED OF IN ACCORDANCE WITH LAWS IN YOUR COUNTRY. CONTACT YOUR COMPETENT LOCAL ADMINISTRATION FOR INFORMATION ON THE RELEVANT LAWS REGARDING DISPOSAL AND RECYCLING IN YOUR AREA



Polymer Technology Systems, Inc.
7736 Zionsville Road
Indianapolis, IN 46268 USA

For assistance with the PTS Connect™ dock

Call PTS Diagnostics Customer Service:

1-877-870-5610 (Toll-free in the US)

+1-317-870-5610 (Direct) +1-317-870-5608 (Fax)

E-mail: inforequest@ptsdiagnostics.com

Website: www.ptsdiagnostics.com

PN 91180 EN



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany