



Customer Bulletin – November 2015

Revised Range Card:

PTS Panels® Ketone Test Strips (Lot Numbers J501 & J502) when using
 PTS Panels® Multi-Chemistry Controls (Lot Numbers MC18 and MC19) – (Cat. No./REF 722)

Summary:

A new range card has been issued when performing controls for PTS Panels Ketone test strips (lot numbers J501 & J502) with PTS Panels Multi-Chemistry Controls (LOT Numbers MC18 and MC19). The new ranges are listed below.

Limited to PTS Panels Ketone Test Strip (Lot Numbers J501 & J502):

The new range card is limited to PTS Panels Ketones test strip (Lot Numbers J501 & J502) when used in conjunction with PTS Panels Multi-Chemistry Controls (Lot Numbers MC18 and MC19) and does not affect other lots of PTS Panels Ketones test strips or PTS Panels Multi-Chemistry Controls.

PTS Panels Multi-Chemistry Controls - Lot Number MC18

OLD

REVISED

		Level 2		Level 2	
		mg/dL	mmol/L	mg/dL	mmol/L
New Range	Ketone (J501)	40-70	3.84 – 6.73	20-45	1.92-4.32
New Range	Ketone (J502)	40-70	3.84 – 6.73	20-60	1.92 -5.77

PTS Panels Multi-Chemistry Controls - Lot Number MC19

OLD

REVISED

		Level 2		Level 2	
		mg/dL	mmol/L	mg/dL	mmol/L
New Range	Ketone (J501)	30-70	2.88 – 6.73	15-50	1.44-4.81
New Range	Ketone (J502)	30-70	2.88 – 6.73	20-60	1.92 -5.77

Questions/Comments?

Please call Customer Service at +1-317-870-5610 (direct) or 1-877-870-5610 (toll-free US only).