



Customer Bulletin – March 2016

Revised Range Card:

PTS Panels® Multi-Chemistry Controls (Lot Numbers MC20 and MC21) – (Cat. No. / REF 721)

Summary:

A new range card has been issued for PTS Panels Multi-Chemistry Controls (LOT Numbers MC20 and MC21). No other lots are affected.

Single Analyte Glucose Range Update (including PTS Panels® eGLU test strips):

The new range card updates the range for glucose when utilizing single-analyte PTS Panels test strips, including PTS Panels eGLU test strips. The updated ranges are provided below and on the next page, with the change highlighted. Updated range cards are also available at www.ptsdiagnostics.com. The previous, obsoleted range cards are also provided for reference.

PTS Panels Multi-Chemistry Controls - Lot Number MC20

PTS Panels® Multi-Chemistry Control* Ranges	Level 1		Level 2	
	mg/dL	mmoL/L	mg/dL	mmoL/L
Lot #: MC20 Expiration Date: 2016 08 31				
Single-Analyte Test Strips				
Cholesterol	130 – 210	3.37 – 5.44	160 – 320	4.14 – 8.29
Triglycerides	110 – 190	1.24 – 2.15	170 – 310	1.92 – 3.50
Glucose/eGLU	40 – 110	2.22 – 6.11	190 – 360	10.55 – 19.98
Ketone	≤ 2 - 20	≤ 0.19 – 1.92	20 – 60	1.92 – 5.77
Multi-Analyte Test Strips				
Cholesterol	120 – 210	3.11 – 5.44	160 – 300	4.14 – 7.77
Triglycerides	80 – 190	0.90 – 2.15	170 – 320	1.92 – 3.62
Glucose	40 – 80	2.22 – 4.44	150 – 330	8.33 – 18.32

*HDL Cholesterol requires a separate HDL Cholesterol Control (Cat. No. 722).
For assistance contact PTS Customer Service @ 877-870-5610 or Fax @ 317-870-5608.
Outside of US @ 317-870-5610. Email: inforequest@cardiochek.com

PS-002519 Rev. 9



PTS Panels Multi-Chemistry Controls - Lot Number MC21

PTS Panels® Multi-Chemistry Control* Ranges	Level 1		Level 2	
	mg/dL	mmoL/L	mg/dL	mmoL/L
Lot #: MC21				
Expiration Date: 2016 10 31				
Single-Analyte Test Strips				
Cholesterol	130 – 200	3.37 – 5.18	170 – 290	4.40 – 7.51
Triglycerides	90 – 180	1.02 – 2.03	170 – 280	1.92 – 3.16
Glucose/eGLU	40 – 110	2.22 – 6.11	190 – 360	10.55 – 19.98
Ketone	≤ 2 - 20	≤ 0.19 – 1.92	20 – 60	1.92 – 5.77
Multi-Analyte Test Strips				
Cholesterol	120 – 220	3.11 – 5.70	160 – 350	4.14 – 9.07
Triglycerides	90 – 190	1.02 – 2.15	160 – 340	1.81 – 3.84
Glucose	30 – 90	1.67 – 5.00	130 – 330	7.22 – 18.32
<p>*HDL Cholesterol requires a separate HDL Cholesterol Control (Cat. No. 722). For assistance contact PTS Customer Service @ 877-870-5610 or Fax @ 317-870-5608. Outside of US @ 317-870-5610. Email: inforequest@cardiochek.com</p>				
				PS-002519 Rev. 9



Previous Range Card Information (Obsolete):

PTS Panels Multi-Chemistry Controls - Lot Number MC20 - OBSOLETE

PTS Panels® Multi-Chemistry Control* Ranges	Level 1		Level 2	
	mg/dL	mmoL/L	mg/dL	mmoL/L
Lot #: MC20 Expiration Date: 2016 08 31				
Single Analyte Test Strips				
Cholesterol	130 - 210	3.37 - 5.44	160 - 320	4.14 - 8.29
Triglycerides	110 - 190	1.24 - 2.15	170 - 310	1.92 - 3.50
Glucose	40 - 90	2.22 - 5.00	190 - 360	10.55 - 19.98
Ketone	≤2 - 20	≤0.19 - 1.92	20 - 60	1.92 - 5.77
LDL Cholesterol	70 - 150	1.81 - 3.89	110 - 200	2.85 - 5.18
Multi-Analyte Test Strips				
Cholesterol	120 - 210	3.11 - 5.44	160 - 300	4.14 - 7.77
Triglycerides	80 - 190	0.90 - 2.15	170 - 320	1.92 - 3.62
Glucose	40 - 80	2.22 - 4.44	150 - 330	8.33 - 18.32
*HDL Cholesterol requires a separate HDL Cholesterol Control (Cat. No. 722). For assistance contact PTS Customer Service @ 877-870-5610 or Fax @ 317-870-5608. Outside of US @ 317-870-5610. Email: inforequest@cardiochek.com				
PS-002519 Rev. 7				

PTS Panels Multi-Chemistry Controls - Lot Number MC21 - OBSOLETE

PTS Panels® Multi-Chemistry Control* Ranges	Level 1		Level 2	
	mg/dL	mmoL/L	mg/dL	mmoL/L
Lot #: MC21 Expiration Date: 2016 10 31				
Single Analyte Test Strips				
Cholesterol	130 - 200	3.37 - 5.18	170 - 290	4.40 - 7.51
Triglycerides	90 - 180	1.02 - 2.03	170 - 280	1.92 - 3.16
Glucose	40 - 90	2.22 - 5.00	190 - 310	10.55 - 17.21
Ketone	≤2 - 20	≤0.19 - 1.92	20 - 60	1.92 - 5.77
Multi-Analyte Test Strips				
Cholesterol	120 - 220	3.11 - 5.70	160 - 350	4.14 - 9.07
Triglycerides	90 - 190	1.02 - 2.15	160 - 340	1.81 - 3.84
Glucose	30 - 90	1.67 - 5.00	130 - 330	7.22 - 18.32
*HDL Cholesterol requires a separate HDL Cholesterol Control (Cat. No. 722). For assistance contact PTS Customer Service @ 877-870-5610 or Fax @ 317-870-5608. Outside of US @ 317-870-5610. Email: inforequest@cardiochek.com				
PS-002519 Rev. 8				

Questions/Comments?

Please call Customer Service at +1-317-870-5610 (direct) or 1-877-870-5610 (toll-free US only).